


Ozonated Saline Set Up

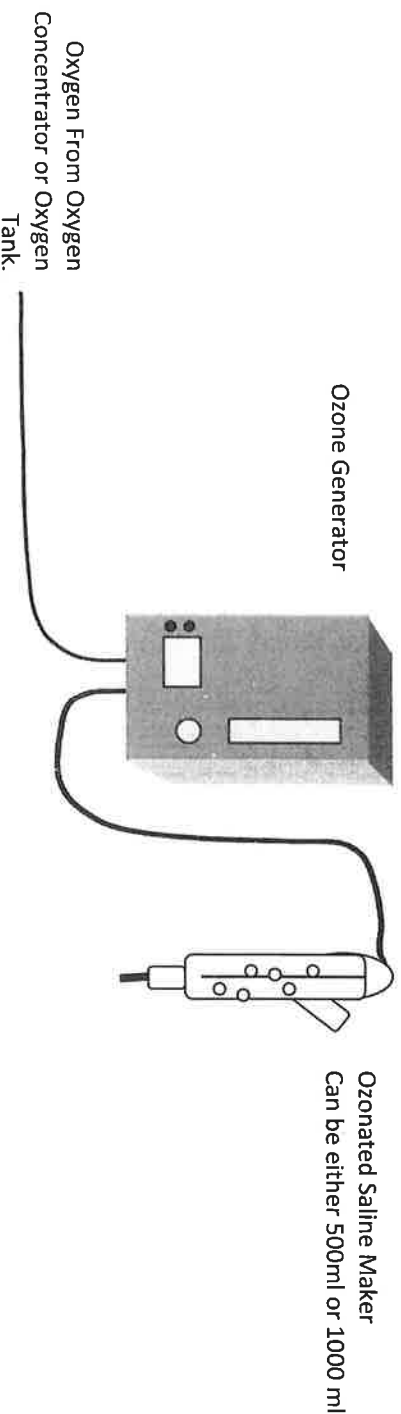

1. Ozone Generator can be mounted to the wall or lying on its back on a table top.
2. Connect the Oxygen line from the ozone generator to an oxygen source (either an Oxygen Tank or an Oxygen Concentrator).
3. Set the ozone generator on level 10 and the oxygen flow rate on 1/8 LPM (Liters Per Minute).
4. Hang the Ozonated Saline Maker (OSM) from a hook on the wall. We discourage the use of an IV Stand however if you are using an IV Stand, ensure that the stand is stable and will not tip over => the OSM is very heavy when full of Saline and therefore a typical IV Stand can topple easily.
5. Remove the glass Ozone Destruct (black inside, red cap on top) and Fill the OSM to the desired level through that right hand hole. The top of the Saline Maker where the ozone line attaches is very fragile, so once you have this in place it is best to leave it there, and use the Ozone Destruct hole (right side) to fill the OSM.
6. Place the Ozone Destruct back on the OSM. Never run the OSM without this in place. Attach the Ozone line from the ozone generator, to the inlet line of the OSM if it is not already attached.
7. Ozonate the Saline for 10 minutes for 500 ml and 15 minutes for 1000 ml. Note that the built in Ozone Destruct Unit protruding from the right side will turn the escaping ozone back into oxygen before it escapes into the room. Ensure that the Glass Ozone Destruct is in always place. This will prevent ozone from entering the room. If you see water droplets inside the Ozone Destruct, remove the destruct, remove the red top, and pour the black pellets out onto a piece of paper towel to dry them. Once dry, put them back into the glass destruct and place the Destruct back on the OSM.
8. Use the saline as desired. If desired, a line can be attached to the exit tube of the Saline Maker, and the stop cock at the bottom of the OSM can be adjusted to the preferred release rate of the saline.
9. Continue the ozonation of the Saline as it exits the Saline Maker to ensure that it remains ozonated as long as possible.

Longevity Resources Inc., 9544 Aurora Place, North Saanich, BC V8L 5V5, Canada

Tel: 250-654-0092 Toll Free Tel: 1-877-543-3398 Fax: 250-654-0093 Toll Free Fax: 1-888-543-3439

Web Site: www.ozonegenerator.com Email: info@ozonegenerator.com